## EU-9167 据付要領書 INSTALLATION PROCEDURES

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This installation procedure is provided for reference in installation of EU-9167.

Tools required: Phillips-type screwdriver. (Prepare them beforehand)

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### CAUTION

- This system must be installed only by the qualified personnel.
- The environmental conditions for the place of installation and the specifications of the power supply must satisfy the requirements stated in the operator's manual.
- Make sure that you will not damage PC board by the static electricity when you handle it.

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Check all items listed below are included in the shipping case.

No.	品 名 Parts Name	外 観 Appearance	個 数 Quantity
1	TVIF 基板 (EP573000**) TVIF board (EP573000**)		1
2	六角サポート :SSD-ARIETTA70 用 (ASB-318E) Spacer :For SSD-ARIETTA70 (ASB-318E)		4
3	六角サポート :SSD-ARIETTA60 用 (BSB-310E) Spacer :For SSD-ARIETTA60 (BSB-310E)		4
4	付属ねじ (CNK3×8 Fe-Ni) Accessory Screws (CNK3×8 Fe-Ni)		8

<b>目次</b>	<b>CONTENTS</b>
-----------	-----------------

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
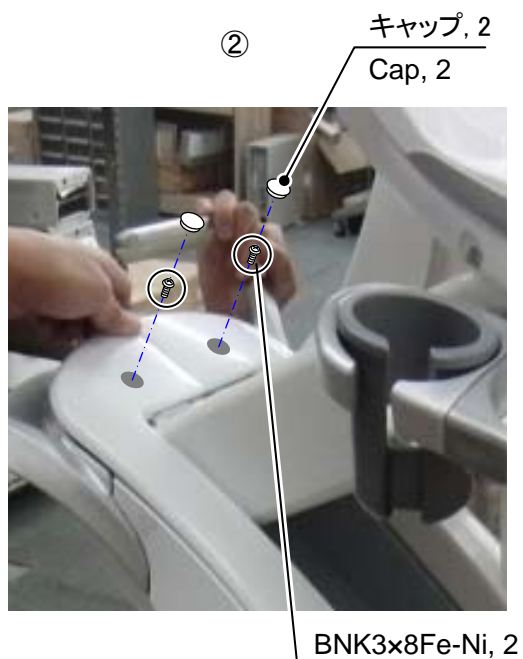
①With the up-down pedal of operation panel, raise the operation panel.

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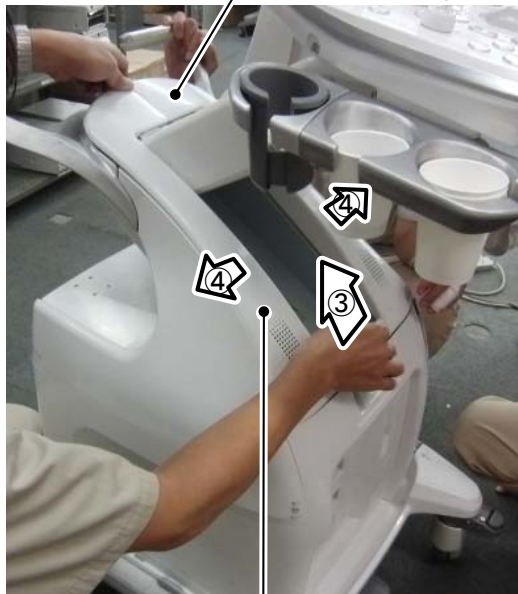アーム開口カバー(左)  
Arm Cover (Left Side)

③Lift the Arm Cover (Left Side) and the Arm Cover (Right Side) together, and remove the lower part of it.

④Separate the Arm Cover (Left Side) and the Arm Cover (Right Side) and remove them.

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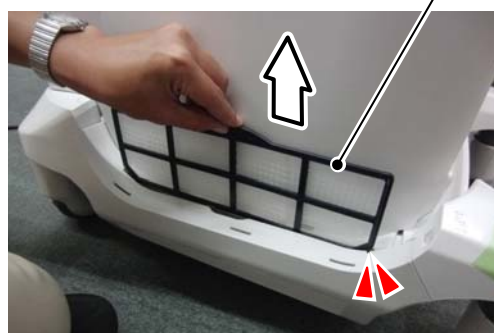
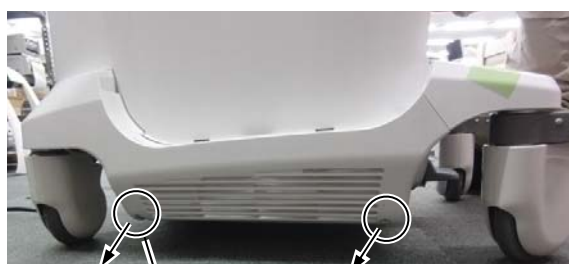
①Pull the Left Side Pocket upward and remove it.

Lift the Dust Filter and remove it.

NOTE: When installing the Dust Filter, pass the protrusion of Dust Filter through the lack of the Left Side Base Cover.

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Left Side Base Cover


②Unfasten 2 screws and remove 3 hooks, then remove Left Side Base Cover.

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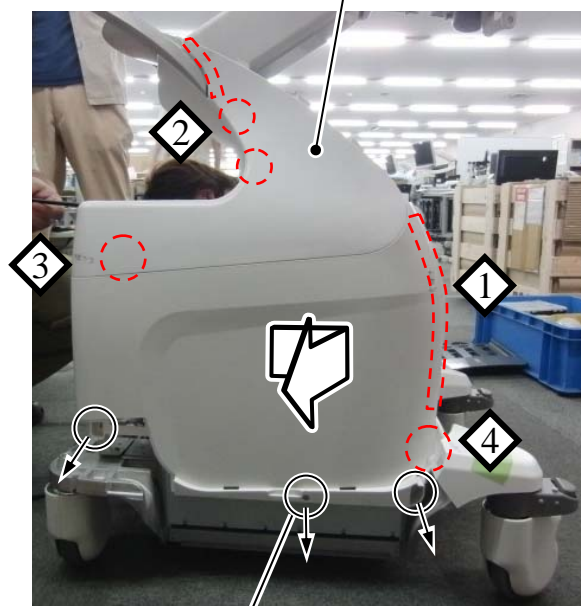
①Unfasten 3 screws.

Slide the Left Side Cover forward and remove the hooks followed the procedures described in figure and remove the Left Side Cover.

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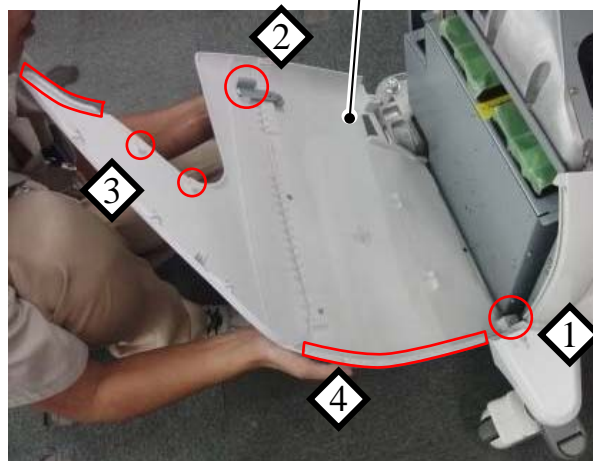
NOTE: When installing the Left Side Cover, insert the hooks followed the procedures described in figure and slide it backward, and mount it.

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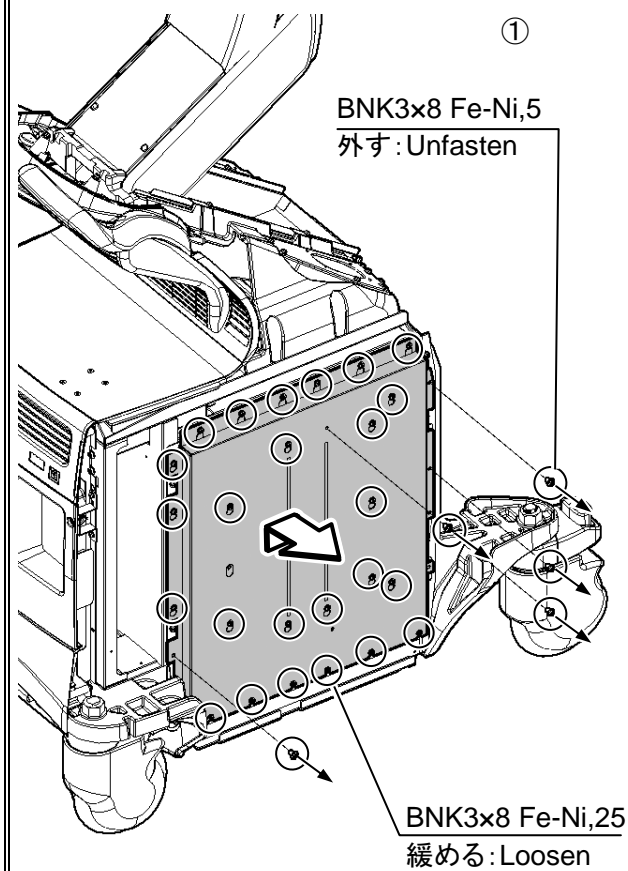①Unfasten 5 screws and loosen 25 screws, then remove the shield cover.

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
Caution: Be careful of handling this unit because it is static electricity sensitive.

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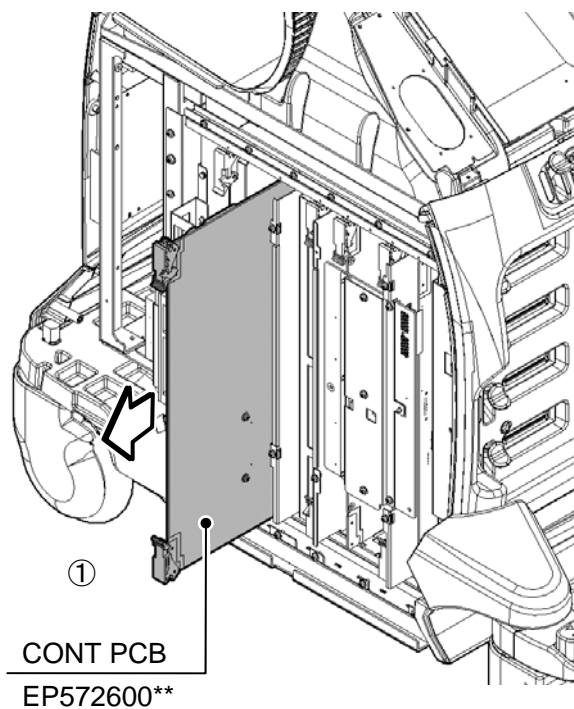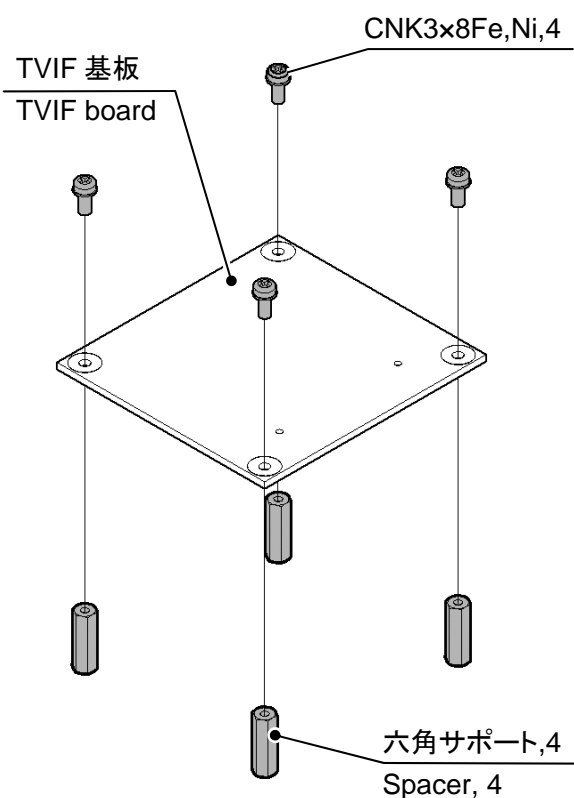
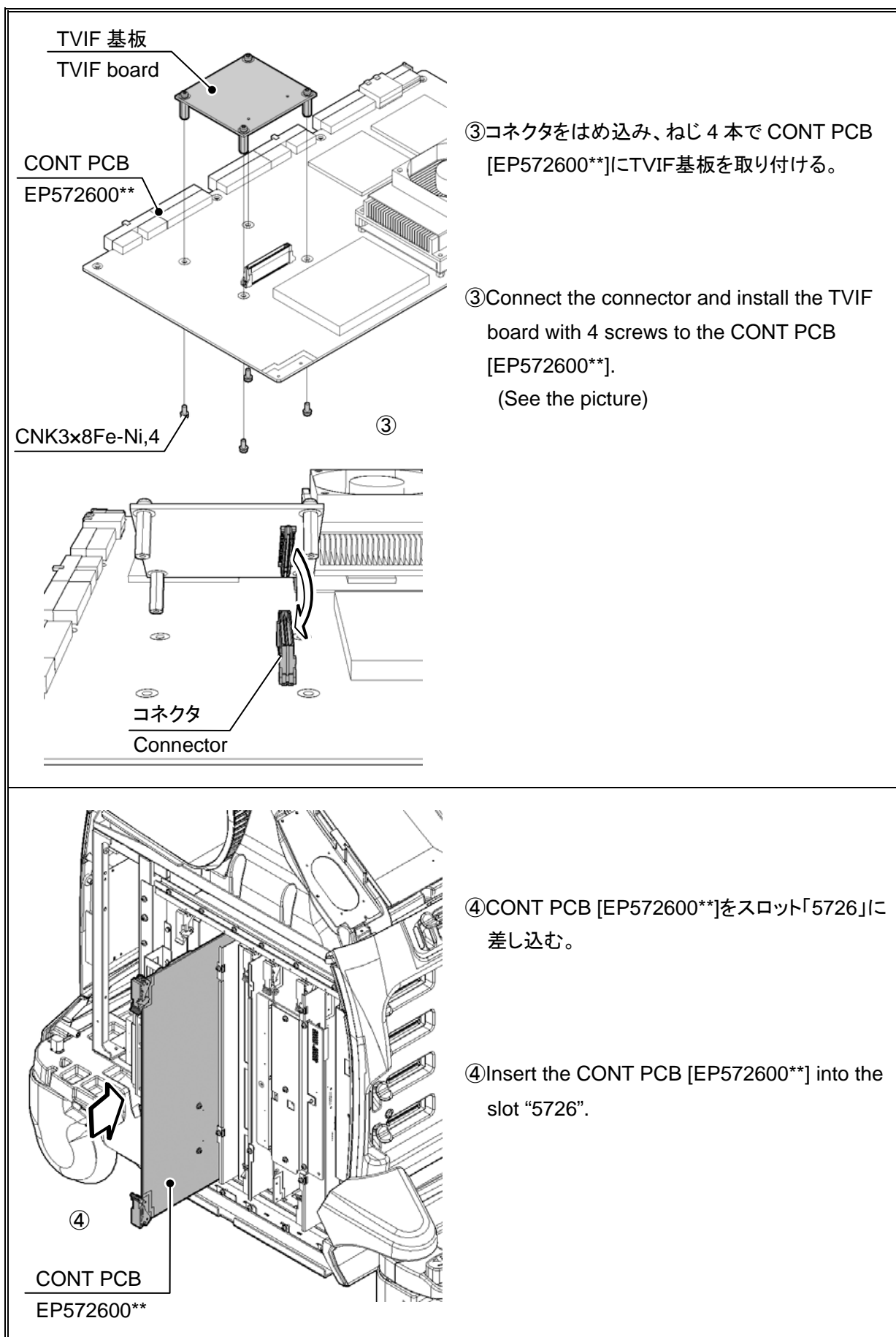
②Install the Spacer with 4 screws to the TVIF board.

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Factory default setting of analog video I/O is NTSC for all the general export equipment. The setting must to be changed to PAL in PAL region.

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#### Cautions

Follow the instruction below. All the data in the equipment such as preset, image data, and patient information are initialized at the time of changing analog I/O setting to PAL. Make sure that full backup is made beforehand.

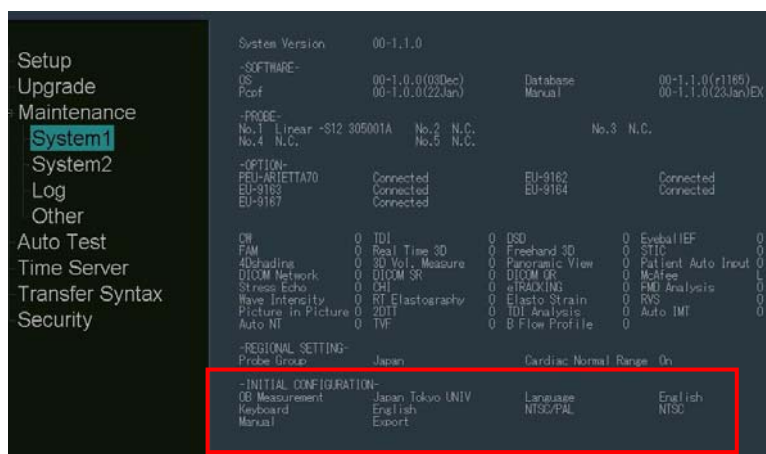
1. Making note of default settings
2. Full backup
3. Change the setting of analog video I/O setting to PAL
4. Restoration of backed up data

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
### < 1. Making note of default settings >

Default setting must be made when changing analog video I/O setting. Make note of current default setting beforehand.

#### [Procedure]

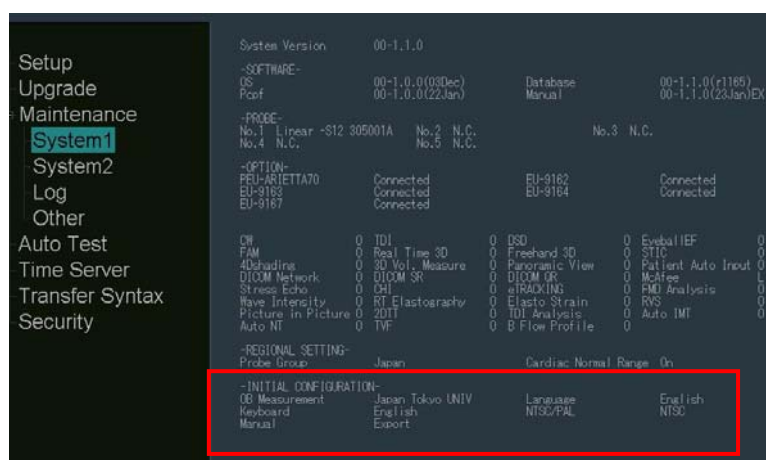
- ① Turn the [POWER] switch on.
- ② After the ultrasound image is displayed, press the [PROBE] switch on operation panel, and press [Preset Setup] switch on touch panel.
- ③ Press Ctrl+S keys on the keyboard to display Password entry dialogue box.  
(Press Ctrl key and S key sequentially when using the Virtual Keyboard.)  
-Move the cursor to Password entry field by trackball and press the [ENTER] switch.  
-Type "moani" into the Password entry field.  
(Turn off the Ctrl key and type Password when using the Virtual Keyboard.)  
-Select the "OK" switch on the dialogue box
- ④ Select "System 1" from the list displayed on the left hand of the screen.
- ⑤ Make a note of "INITIAL CONFIGURATION" shown in the image below.

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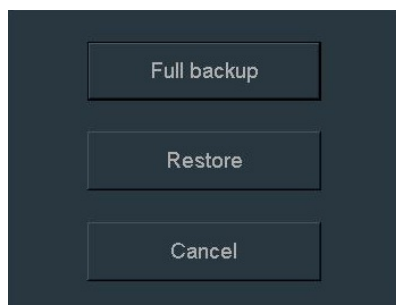
### <Full Backup>
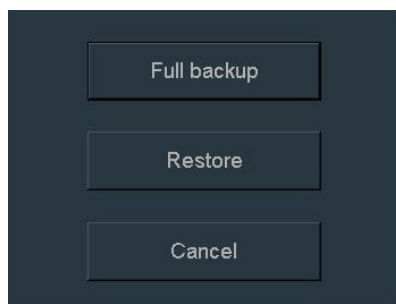
#### [Procedure]

- ① Turn the [POWER] switch on.
- ② After the ultrasound image is displayed, press the [PROBE] switch on operation panel, and press [Preset Setup] switch on touch panel.
- ③ Select [Preset Control] switch on Preset Setup display.
- ④ On the Preset Control display, select the destination of the backup to save in the Media.
- ⑤ The dialogue below is displayed after the “Full Backup/Restore” switch is selected. Select “Full backup” switch to start backup.


- ⑥ A dialogue “The backup is successful” is displayed after successful completion. Select [OK] switch, and select [Cancel] switch to close the dialogue.
- ⑦ Select [Close] switch on the lower right of the screen, and select [Close] switch on the lower right of the screen again, to return to the ultrasound display.

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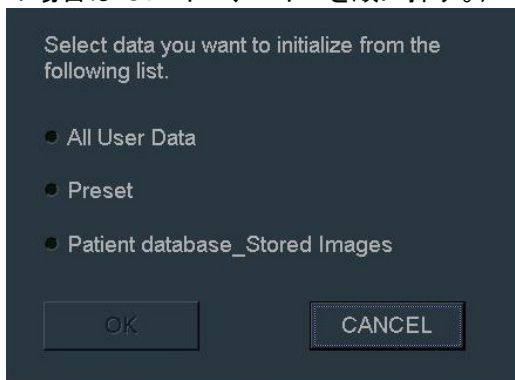
### < Change the setting of analog video I/O setting to PAL >
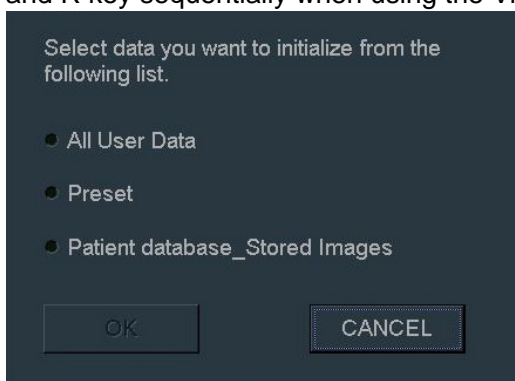
#### [Procedure]

- ① Turn the [POWER] switch on.
- ② After the ultrasound image is displayed, press the [PROBE] switch on operation panel, and press [Preset Setup] switch on touch panel.
- ③ Press Ctrl+R keys on the keyboard to display the dialogue below. Select "All User Data", and select [OK] switch on the dialogue.  
(Press Ctrl key and R key sequentially when using the Virtual Keyboard.)


- ④ Select [OK] switch on the confirmation dialogue and the equipment starts to shut down automatically. After the shutdown procedure is completed, press [POWER] switch to restart the equipment.
- ⑤ Initial Configuration dialogue is displayed after the restart. Input the setting noted on <Making note of default settings> procedure ⑤. Set NTSC/PAL to PAL.
- ⑥ Select [Exit] switch on "Initial Configuration" dialogue, and the equipment starts to shut down automatically.

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
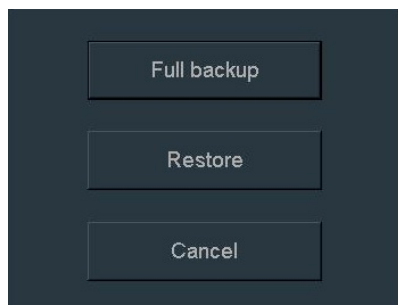
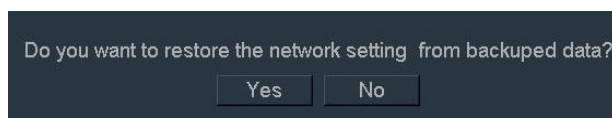
< Restoration of backed up data >

[Procedure]

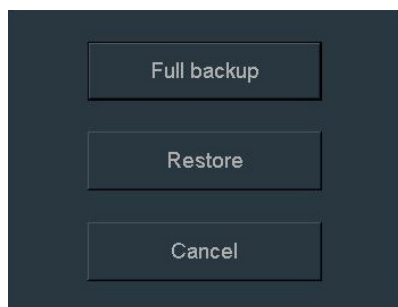- ① Turn the [POWER] switch on.
- ② After the ultrasound image is displayed, press the [PROBE] switch on operation panel, and press [Preset Setup] switch on touch panel.
- ③ Press [Preset Control] switch on Preset Setup display.
- ④ On the Preset Control display, select the device of backed up data in the Media.
- ⑤ The dialogue below is displayed after the “Full Backup/Restore” switch is selected. Select “Restore” switch to start restoration.

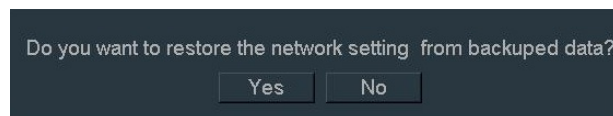- ⑥ Select data to be restored on “Please select database” dialogue, and select [OK] switch.
- ⑦ Go on selecting following the confirmation dialogue displayed on the screen.
- ⑧ The restoration starts after selecting “Yes” or “No” switch on the dialogue below. The dialogue “The restore is successful” is displayed after successful completion.

- ⑨ Select [OK] switch, and the equipment starts to shut down automatically. After the shutdown procedure is completed, press [POWER] switch to restart the equipment.

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##### < In case of using Y/C In >

- ①Confirm the following items.

- System boots normally.
- Displaying image from external (NTSC/PAL) device under EXT condition.
- In the state of EXT, the input image is displayed on the dotted line area of Figure 3-1.
- No input or not detected signal, blue square is displayed on dotted line area of Figure 3-1.

NOTE: For the details of operation, refer to the instruction manual.

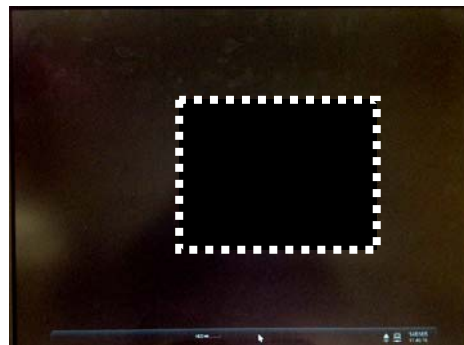
Figure 3-1: Monitor of EXT state

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- ①Install the cover with the reverse way of Removing of covers.

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
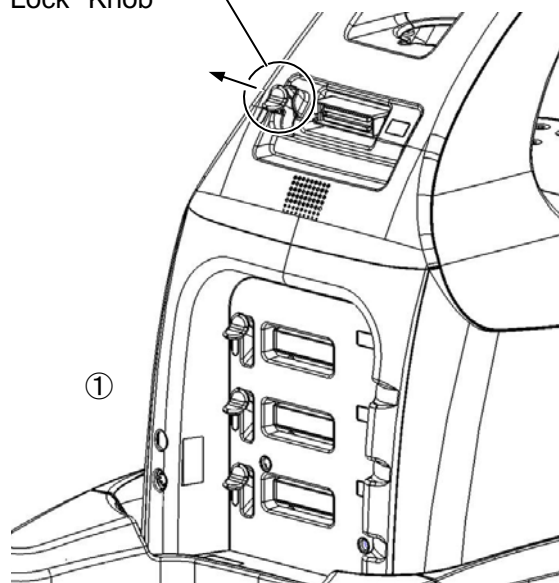
①Raise the operation panel to the highest  
position and remove 4 Lock Knobs.

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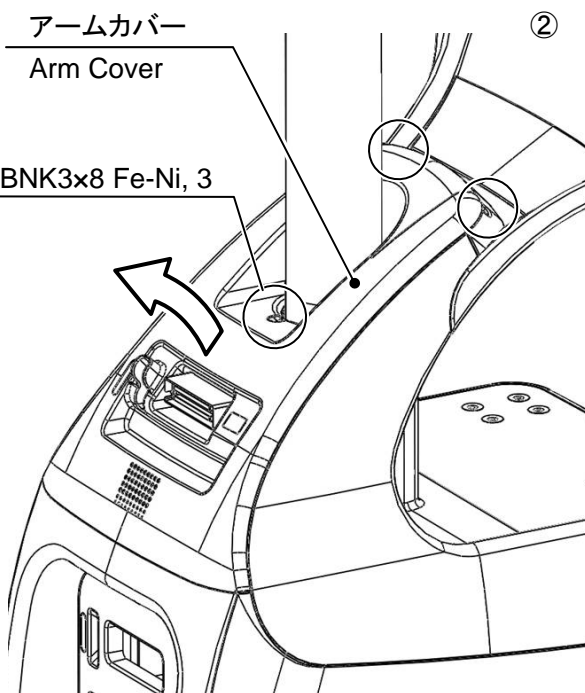
②Loosen 2 screws from side of the handle and  
1 screw from side of the arm, then remove  
the Arm Cover.

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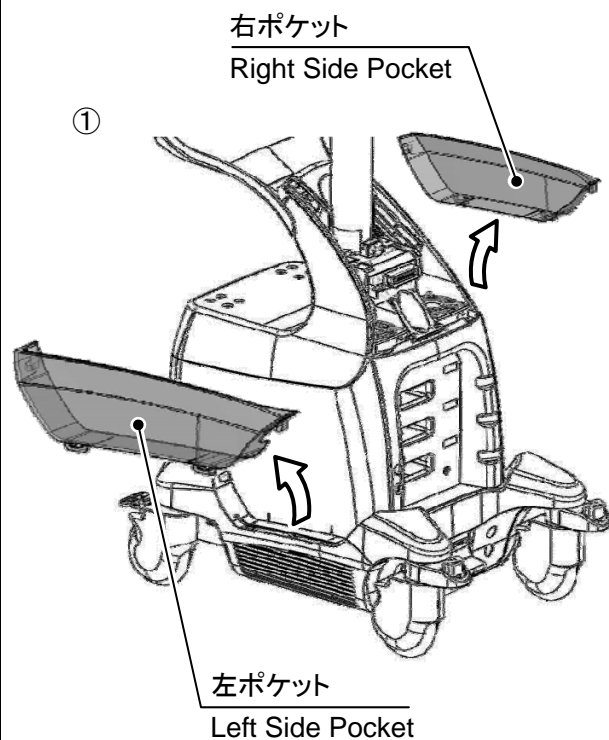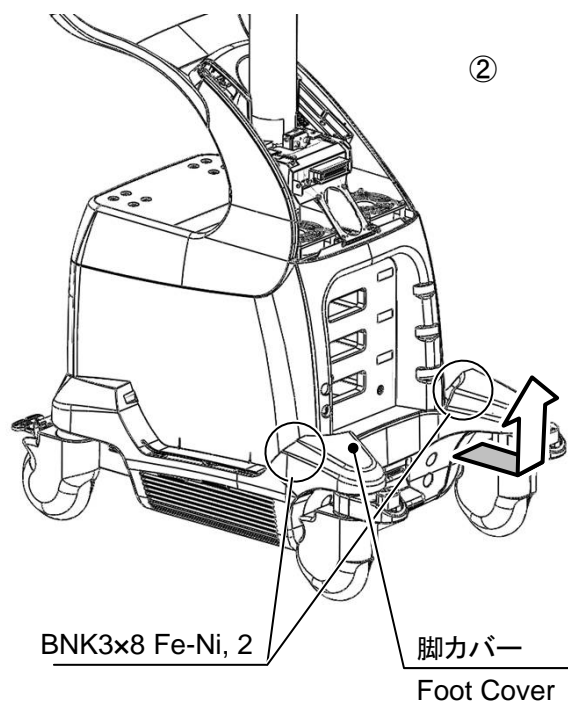
①Pull the Left Side Pocket and Right Side Pocket upward and remove it.

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
②Loosen 2 screws and slide the Foot Cover to the front of equipment. Lift the Foot Cover, and remove it.

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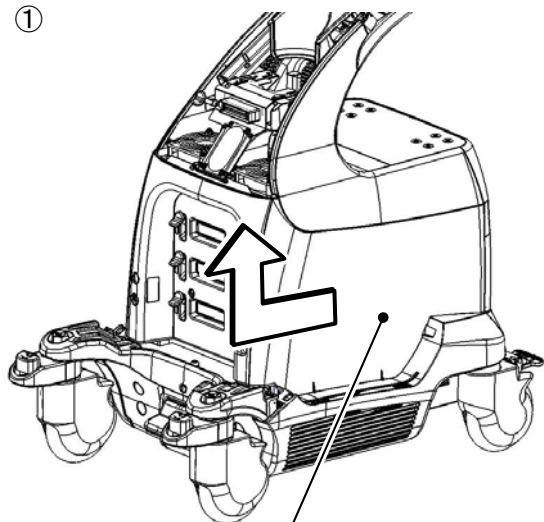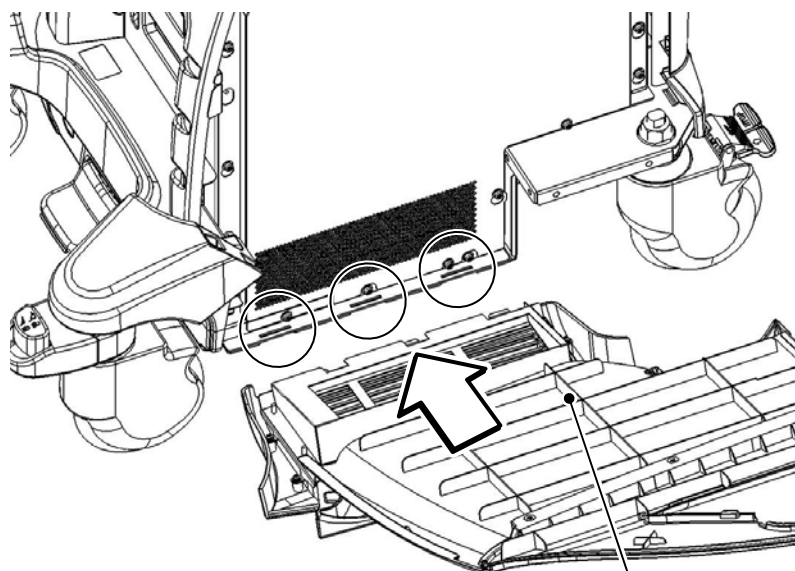
① Slide the Right Side Cover to the front of the equipment and remove it with lifting.

※ When installing the Right Side Cover, insert the hooks of the Right Side Cover to slits of the equipment followed the figure and slide it backward, and mount it.

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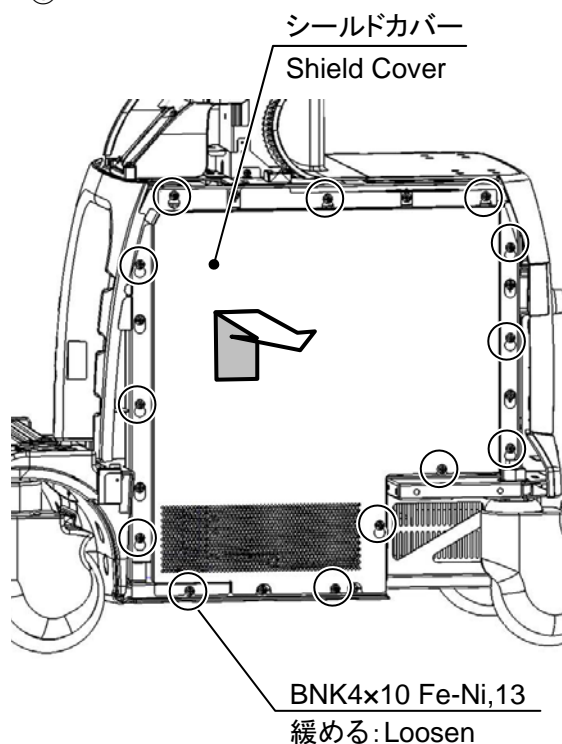
①Loosen 13 screws at the Shield Cover, slide the Shield Cover upward and remove it.

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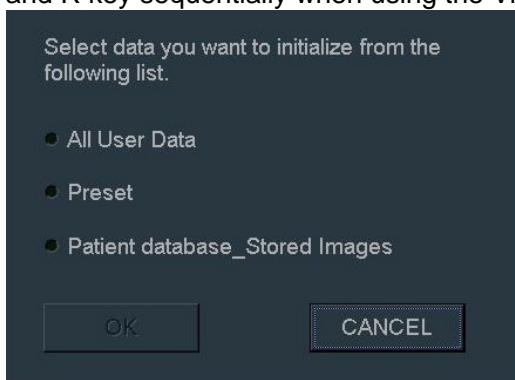
- ④ Select [OK] switch on the confirmation dialogue and the equipment starts to shut down automatically. After the shutdown procedure is completed, press [POWER] switch to restart the equipment.
- ⑤ Initial Configuration dialogue is displayed after the restart. Input the setting noted on < Making note of default settings > procedure ⑤. Set NTSC/PAL to PAL.
- ⑥ Select [Exit] switch on "Initial Configuration" dialogue, and the equipment starts to shut down automatically.

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